

DCAR 系列培训

MAC 800 心电分析系统

闵昊

28, 8, 2023



imagination at work



课程导航:

1. MAC 800 安装指导
2. MAC 800 维修常见故障诊疗与排除

MAC 800 安装指导

外观检验

- 检验外包装箱是否有碰撞、破损的痕迹。打开包装，检验机器，看是否外观有损坏。假如发觉，请立即告知GE企业的销售代表，并拍照取证；
- 对照装机清单，查看应有的配件是否齐全，电源线，患者缆线（导联线），导联夹，吸球，锂电池，SD卡。假如有关选配功能，查看是否有关配件齐全；

外观检验（前面板）



外观检验（侧视）


ECG Signal Input Connector

- Connects the patient cable to the device.



Secure digital (SD) card
for External Storage

外观检验（后视）

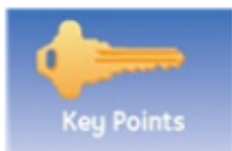

A = Standard RJ-11 jack

B = LAN connector slot

C = Universal Serial Bus


D = COMM Port

E = AC Power Connector


F = Equipotential
Grounding Lug

G = Carry Handle

H = Fuse Holder

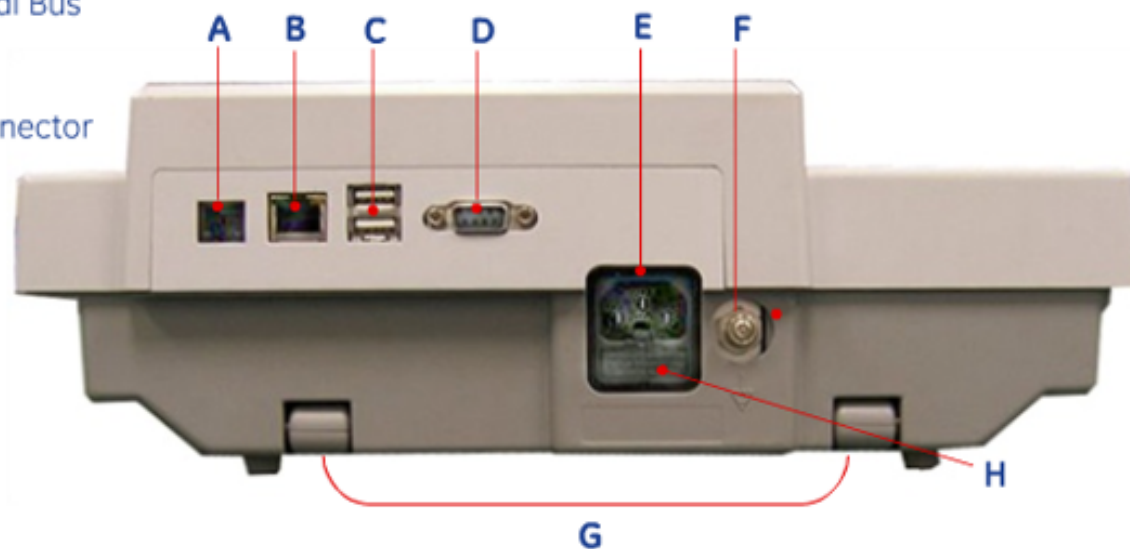

外观检验（底面）

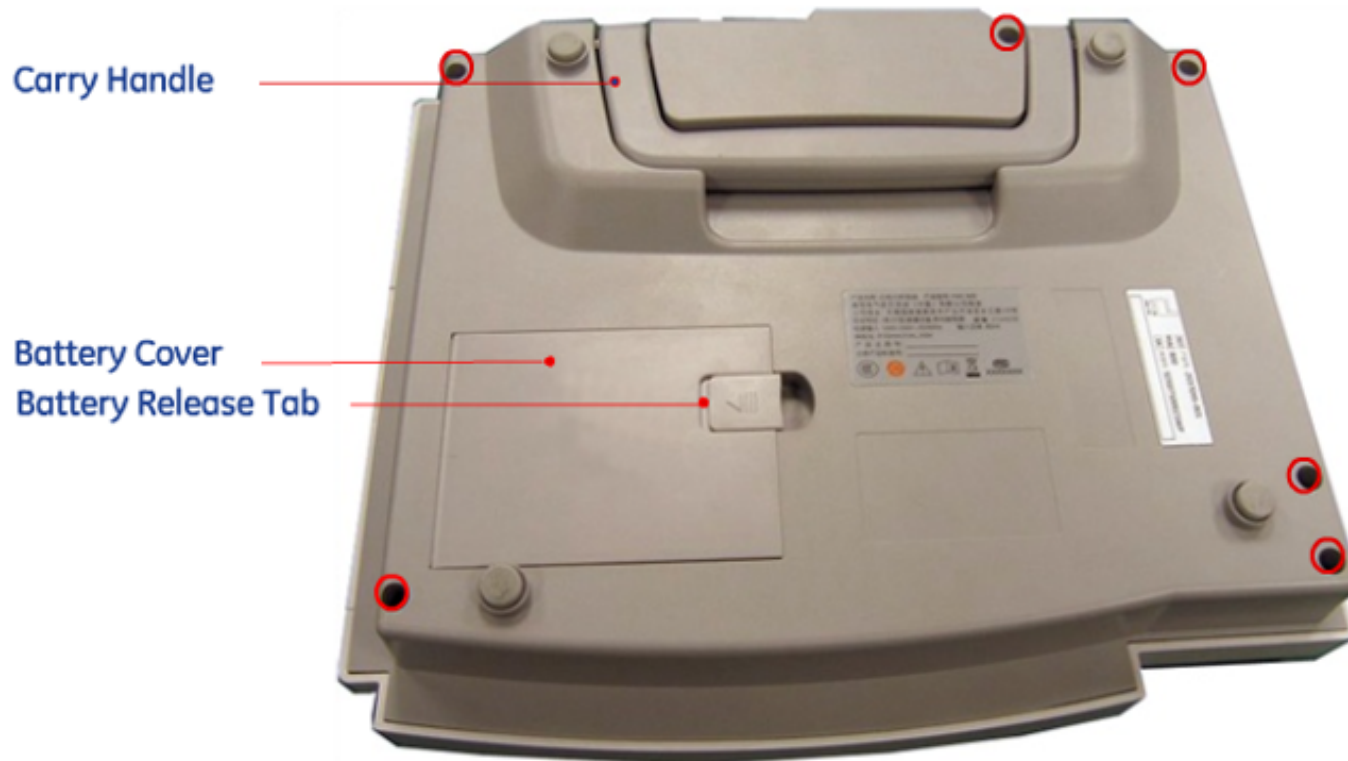

电池

- User replaceable
- 7.2V@ 4.5 AH \pm 10%
- Rechargeable Lithium Ion
- Capacity: 1000 single page reports (typical) or
- 2 hours continuous display (without printing)



键功能检验



装机流程

1. Inserting the battery.
2. Connecting the AC power adapter.
3. Connecting leadwires.
4. Inserting paper.
5. Connecting the LAN.
6. Connecting the external laser printer.
7. Turning on the unit.
8. Configuring the system.
9. Testing the equipment.



Insert the battery

1. Gently turn unit over and find the empty battery compartment.
2. Insert the battery as shown.



Connect the AC power adapter

1. Connect the female end of the unit's power cord to the AC power connector on the back of the unit.
2. Plug the male end of the unit's power cord into an AC outlet.
3. Check the Power LED to make sure the unit is receiving power from the AC outlet.



3
Power LED



Connect leadwires

1. Assemble the leadwires and adapters.
2. Connect the leadwires to the front of the acquisition module.
3. Connect the acquisition cable to the MAC 800 system.



以上内容仅为本文档的试下载部分，为可阅读页数的一半内容。如要下载或阅读全文，请访问：<https://d.book118.com/115122242323011330>